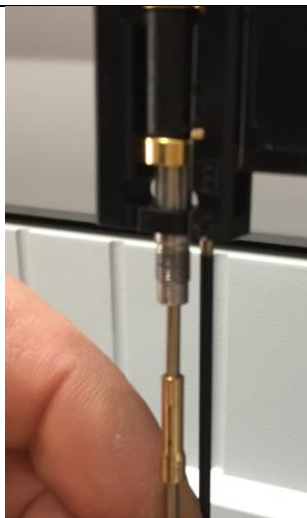
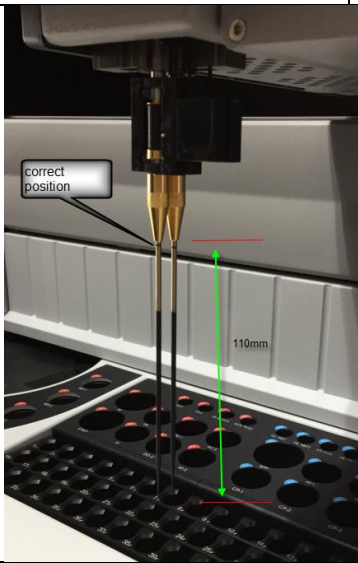


How to replace a needle on Coatron A6 Plus\5000

		
Step1: Open nut and move needle at little down. Then remove the clamping.	Step2: Open the tubing guide	Step3: Shift out needle+tube
		
Step4: Remove tube from needle and cut end	Step5: Widen the tubing with tool	Step6: Fix tube with tool and shift it over the needle
		SERVICE 1 PRINT REPORT 2 ADJUST XYZ 3 ADJUST TEMPERATURE 4 CHECK OPTIC 5 CHECK ROBOTICS 6 MOVE CUVETTES 7 CLEAN NEEDLE 8 REPLACE RINSE TANK 9 ADJUST MOTOR 10 MICROTUBE
Step6: Shift in needle. Watch, that COAX cable do not block the needle	Step7: Tight the golden nut. The needle must be 110mm visible. See the black ring mark on needle	Step8: Make fine adjustment as described in the manual