



**MEDICAL INSTRUMENTS,
PRODUCTION + TRADING GMBH**
Web: www.teco-gmbh.com
Email: info@teco-gmbh.com

**DIESELSTRASSE 1
D-84088 NEUFAHRN NB**
TEL: #49 8773 70780 10
FAX: #49 8773 70780 28

Malfunction Coatron A6Plus: Blocking cuvettes


Error description

After measurement the used cuvette is blocking at output position and is not wasted into container. Afterwards the cuvette must be removed manually to continue system operation.

Corrective Actions:

Installation of new part

CAT = 21 023 05 Cuvette output rail

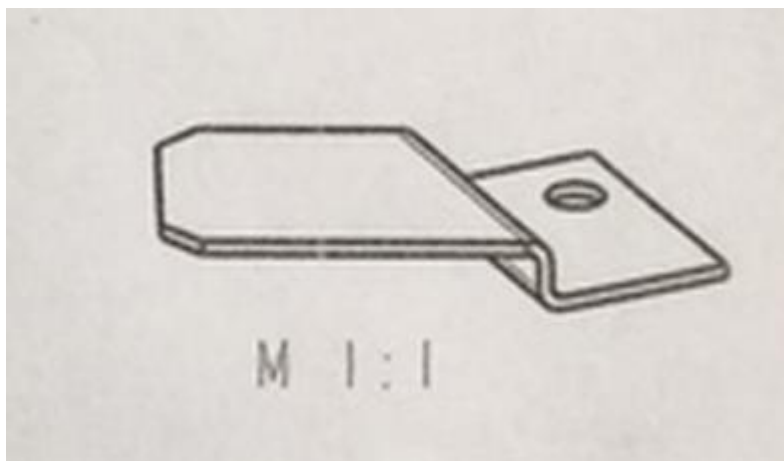

Related systems

Since Jun-2017 the part is installed on all new Coatron A devices

Service action

Coatron A6plus and 5000 with serialnumber xxxx-xxxx-7049 or below must upgrade