

Service Action number	20171110 Coatron-5000
Date of action	21.03.2018

Related products		Coatron A6+	Coatron 5000
	S/N	??	??
	Date of production	Before March, 2018	Before March, 2018
System problems	P1) valve XC+	Get leaky before 25.000 tests	
	P2) cuvette position not accurate	Sometime cuvette don't stop at optic	
	P3) Needle damaged during XYZ adjust	Sometime motor moves when needle is inside of cuvette during ADJUST XYZ	
	P4) Needle crash	Installing control plasma into micro tube cause a needle crash during level sensing	
Service Action	A1) Update firmware	1.03.05a or later	6.03.05a
	A2) Update hardware	Install service kit	Install service kit

Service Kit Coatron-5000 CAT ??

including

1x	23 816 00	valve XC3+
1x	23 033 00	cuvette shifter , Rev-C
2x	23 067 05	Needle A6P
1x	23 063 01	Tubing complete with CAP+Filter

