
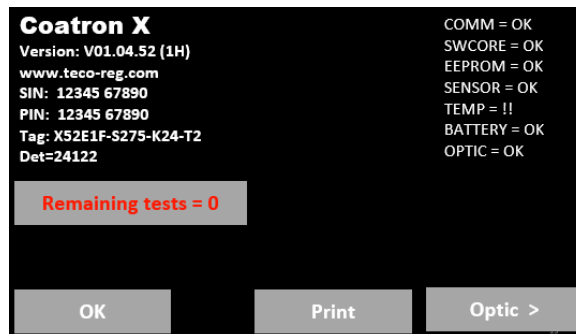



How to activate cuvettes on Coatron X

1) Login to Teco Ticket system www.teco-reg.com

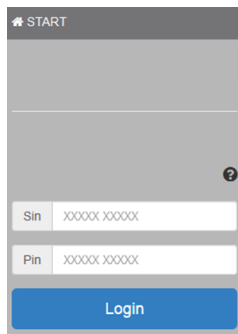

Enter SIN and PIN of device, shown on instrument license plate, system information screen or instrument ID-card.


2) Input Voucher

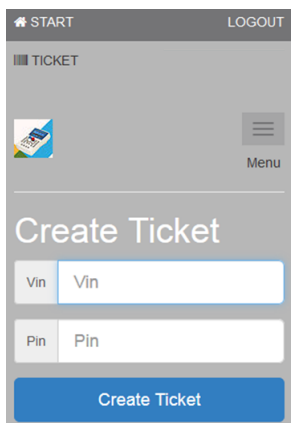

Enter VIN and PIN of voucher




Voucher is inside of cuvette box

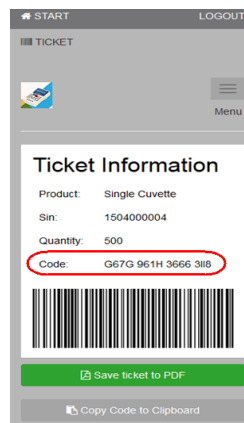

Transfer the code to Coatron X

3) Transfer ticket code to instrument

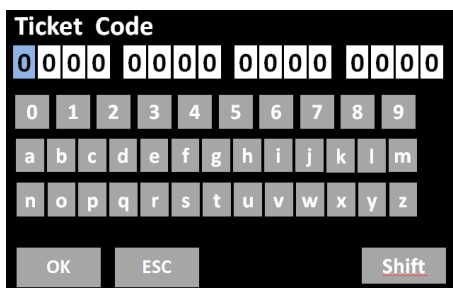

Open info screen (touch blinking RED LED) and then "Remaining tests=0". The code can be transferred by manual input, barcode scanner or TECAM SMART software

<https://www.teco-medical.com/downloads.html>



User manual



Video instructions



Invalid ticket



Coatron X support