



How to replace the Coatron-X display

For instruments:

Coatron X Series and all variants

For sparepart:

REF 25 010 xx

Required tools:

Screwdriver Torx T10; USB A-B cable

Procedure:

1. Removal and replacement

Follow this video instruction



https://www.teco-medical.com/downloads.html?group=Coatron_X&category=&search=display

2. Firmware Update

Identify the type of display (see next page). In case of “F” , install firmware “tag /F” otherwise select firmware “tag /B”.

Follow the update instruction.



https://www.teco-medical.com/downloads.html?group=Coatron_X&category=&search=update

3. Reboot instrument

Instrument may reset to factory default. Check date/time, temperature in cuvette and the test calibration. Afterwards instrument is ready to use.


Identify type of display and correct firmware

Production of first unit with new display type "B"

SIN=01xxx-03850	2023-05-17
SIN=02xxx-03716	2023-24-04
SIN=03xxx-xxxxx	no instruments with type ,B'
SIN=05xxx-04559	2024-07-31
SIN=11xxx-04461	2024-05-24
SIN=15xxx-04497	2024-06-12



old display Type "F"

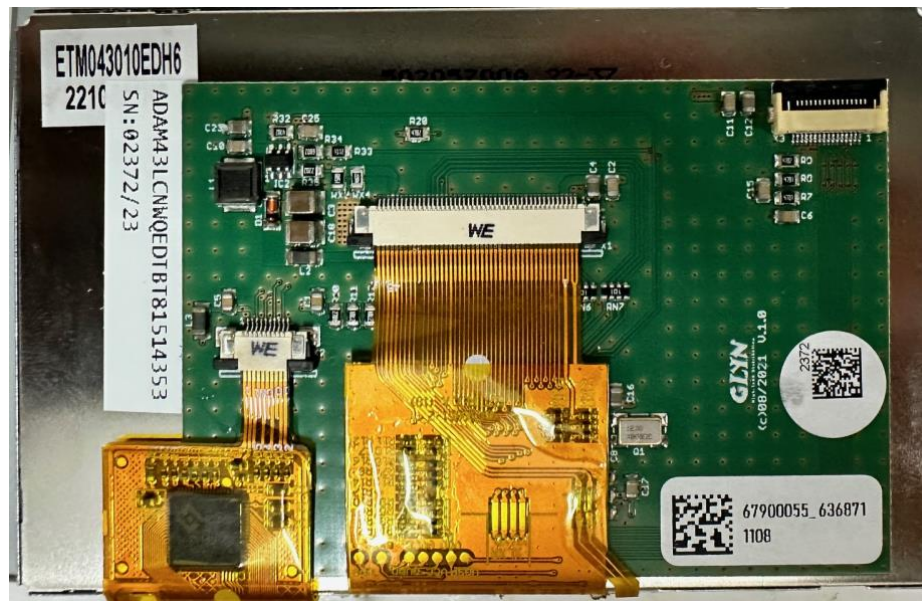

new display Type "B"