



**MEDICAL INSTRUMENTS,
PRODUCTION + TRADING GMBH**

Web: www.teco-gmbh.com
Email: info@teco-gmbh.com

**DIESELSTRASSE 1
D-84088 NEUFABRN NB**


TEL: #49 8773 70780 10
FAX: #49 8773 70780 28

How to reset Coatron-X to factory default

Procedure:



WELCOME SCREEN +

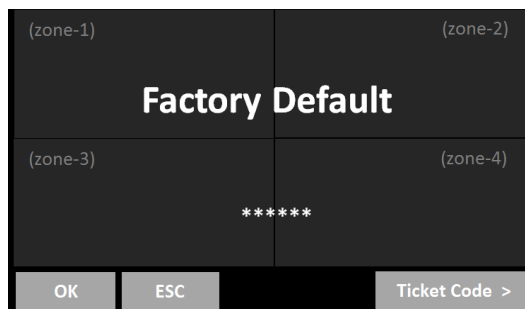

FACTORY DEFAULT

How to reset to factory default:

1. Press 3sec to spinner during boot up
2. Select "OK" Switch on device and switch to home screen
3. Confirm with "OK" again the start the reset



Date, temperature and test calibration must be adjusted after a factory reset !

Default values:

- Temperature sensor = 51000
- Mixer = 1;
- Language = EN;
- Double determination = OFF;
- Auto PID = ON;
- Countdown = OFF;
- All results stored on board are deleted
- All test calibration data are reset to default