



**MEDICAL INSTRUMENTS,
PRODUCTION + TRADING GMBH**

Web: www.teco-gmbh.com
Email: info@teco-gmbh.com


**DIESELSTRASSE 1
D-84088 NEUFABRN NB**

TEL: #49 8773 70780 10
FAX: #49 8773 70780 28

Malfunction Coatron X: Clock error

Error description:

Coatron reset the system date to 1.1.2020 12:00 and report error "Clock" + "System not ready!"

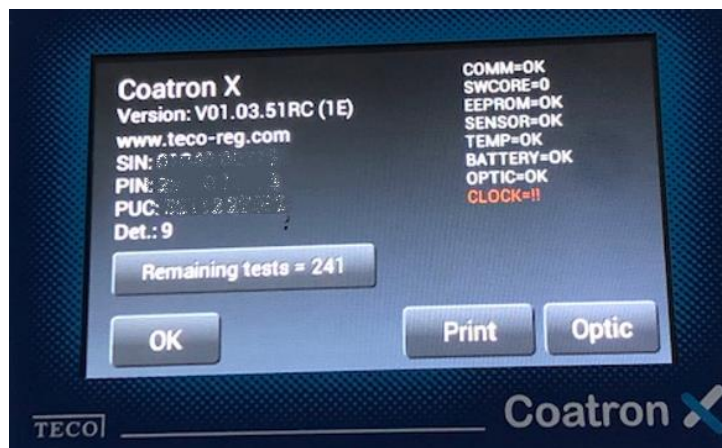

Risk of health:

None. System can't be used.

Reason:

Software failure with version 1.03.50

Recommended service actions:

Quick Fix:

Set the system date back to February 1st 2020

Final Fix:

Update to software version V1.03.51 or later. The latest version can be downloaded here

<https://1drv.ms/f/s!AoYVax1r2TZdZ5rkjiVSoYHundQ>

6.10.2020