




**MEDICAL INSTRUMENTS,
PRODUCTION + TRADING GMBH**
Web: www.teco-gmbh.com
Email: info@teco-gmbh.com


**DIESELSTRASSE 1
D-84088 NEUFAHRN NB**
TEL: #49 8773 70780 10
FAX: #49 8773 70780 28

Malfunction Coatron X: System displays optic error

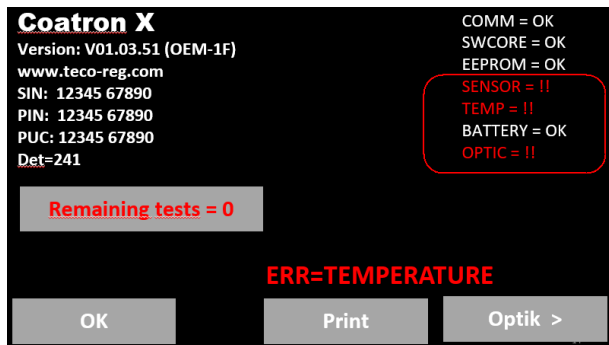

Multi errors (SENSOR+TEMP+OPTIC) are an indicator for ESD (electro static discharge).

- disconnect power for 1min
- remove all cuvettes
- restart device
- check optics again

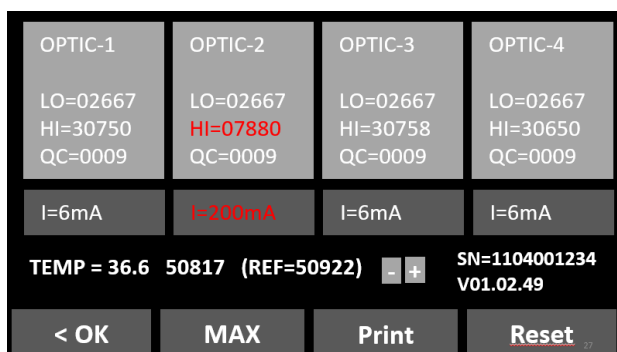

Optic channel-2 is out of range

- Optical channel is blocked -> clean optics
- LED is defect

25.10.2021