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[en] No liability is assumed, if this product is not purchased from any other than the manufacturer or an authorized distributor.

A. General use

A.1 This device must be installed at professional laboratories where a quality management system is established and good laboratory practice (GLP) is applied. The laboratory must follow local regulations about operating medical products. **A.2** This device must be operated by a professional user only. It is not intended for point of care or home testing. **A.3** Read user manual in its entirety prior first operation. **A.4** Install device according to electrical and environmental conditions stated im chapter "Installation" of user manual. **A.5** Operate the instrument as intended and instructed in the User Manual. **A.6** Follow always product labeling and manufacturer's recommendations. **A.7** Use only materials, consumables and (spare)parts such as reagent or cuvettes, which are intended or recommended for use on this instrument. Always contact manufacturer or local authorized distributor in case of doubts.

A.8 Do not use materials after their expiry date or assigned shelf life. **B. Laboratory use** **B.1** Check correct function of instrument by performing quality control before running a series of patient samples or after test calibration. **B.2** Do not operate the instrument after spilling reagent or liquids into the analyser. Contact local authorized distributor for servicing. **B.3** Never use cuvettes more than once. Do not wash cuvettes. They are for single use only. **C. Risk of infection** **C.1** Consider all surfaces and materials, which might be in contact with plasma or other biological liquid as potentially infectious. **C.2** Avoid direct contact with potential infectious materials or surfaces by wearing appropriate protective clothing. **C.3** Follow the laboratory hygienic procedures during and after completion of work. **D. Maintenance and Service** **D.1** Only an authorized customer service may carry out any servicing. **D.2** Decontaminate instrument before doing any servicing or shipment. **D.3** Recycle instrument according to WEEE or local regulations for electronic equipment.

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