

21 510 00

Clean Solution / Clean A

TECO

Information of Use

The Clean Solution (Clean A) is intended as a general laboratory article for a wide range of uses in laboratories. It is also suitable for use in automatically coagulation analysers which are used for in vitro diagnostics tests.

TECO recommend it to remove any residues from needle probe on following TECO instruments: **Coatron A4, Coatron A6, Coatron A6 Plus: Coatron 1800 / 3000 / 5000.**

Contents & Composition

Product	Clean Solution (Clean A)
Cat.No.	21 510 00
Content	1 x 500 ml

Ingredients: 99,6% acidic solution (pH 1.1±0.5) with detergent (slight foaming); generally no hazardous to water.

Preparation and waste information

Ready to use. The solution should be used once only.

Storage and

Avoid freezing, best storage temperature (-18 – 25°C)

Stability

See individual Expiry date on each box.

Precautions

General laboratory safety measures for use of chemical solutions. Avoid direct contact with eyes and skin.

Warranty

This product is warranted to perform in accordance with its labelling and literature. TECO disclaims any implied warranty of merchantability or fitness for any other purpose, and in no event will TECO be liable for any consequential damages arising out of aforesaid express warranty.

Product picture (example)



Clean Solution
(both with Hologram)

Clean A

Outer Package

no specific outer package of single unit
Shipping within total packaging

Safety data sheet: sds-id.com/100202-6

Symbols key (if applicable):

 Temperature	 Manufacturer	 LOT LOT Number	 REF Catalogue Number
---	--	--	--