24 100 00 Single Cuvettes - 500

24 200 00 Single Cuvettes - 5000

TECO

Information of use

The Cuvette as general laboratory article is suitable to hold a reaction mixture for use in optical analyzers which are used in laboratories for in vitro diagnostic tests.

The cuvettes are ready for immediate use. They have unlimited shelf life if stored at 0 - 50°C.

Content

Product	Single Cuvettes	Single Cuvettes
Cat.No.	24 100 00	24 200 00
Content	500 pcs (500 det.)	5000 pcs (5000 det.)

The Cuvettes can be used with Coatron X Analyzers.

For application with semi-automated Coagulation System Coatron X, each box contains a voucher label with a VIN and PIN code to generate a ticket on the web-based registration page (www.teco-reg.com).

The ticket information (VIN/PIN) must be entered for the respective device to release the number of tests for this device only. The (VIN/PIN) can only be used once per unit.

Precautions and waste information

The Cuvette should only be used once in analyzers. To prevent contamination (sample/reagent), it is advised to avoid contact with skin and eyes. Suitable protective clothing and gloves are recommended.

Please also note the disposal of components in accordance with local regulations for infectious material.

Material: pure, clear Polystyrol (PS)
Maximum volume should be less than ~ 500µL
Minimum volume: 75 µL
Dimensions max.: Ø11,5 mm x 24 mm

24 100 00

Example Picture of the package – 500 Single Cuvettes

Packaging:

1. Log Bag, Dim.: (mm) 165 x 295 x 0,05, Mat.: LDPE;
2. Card Box, Dim.: (mm) 252 x 104 x 65
3. Paper Sleeve with Identification and Information

Voucher

Single Cuvettes

VIN: 71101 XXXXX

PIN: 19823 78881

500

Example of Voucher – 500 Single Cuvettes/Pack

24 200 00

Example Picture of the package – 5000 Single Cuvettes

Packaging:

1. Log Bag, Dim.: (mm) 600 x 400 x 0,05, Mat.: LDPE;
2. Card Box, Dim.: (mm) 400 x 250 x 150
3. Label with Identification and Information

Voucher

Single Cuvettes

VIN: 71200 XXXXX

PIN: 19823 78881

5000

Example of Voucher – 5000 Single Cuvettes/Pack

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