80 521 10 Cuvette block, (4pos/ea)

Information of use

The Cuvette block as general laboratory article is suitable to hold a reaction mixture for use in optical analyzers which are used in laboratories for in vitro diagnostic tests.

The cuvettes are ready for immediate use. They have unlimited shelf life if stored at 0 - 50°C.

Content

Product	Cuvette block	1000 pcs
Cat.No.	80 521 10	
Content	50 bundles (= 20 blocks 4 pcs.)	4000 det.

The Cuvettes can be used with Coatron A (1800) Analyzers.

Depending on configuration of IVD analyser instrument a cuvette "activation"-code could be necessary. In this case the box contains an appropriate Voucher with a VIN and PIN code to generate a ticket on the web-based registration page (www.teco-reg.com).

The ticket information (VIN/PIN) must be entered for the respective device to release the number of tests for this device only. The (VIN/PIN) can only be used once per unit.

Precautions and waste information

The Cuvette should only be used once in analyzers. To prevent contamination (sample/reagent), it is advised to avoid contact with skin and eyes. Suitable protective clothing and gloves are recommended.


Please also note the disposal of components in accordance with local regulations for infectious material.

Material: pure, clear Polystyrol (PS)

Maximum volume per Single use Unit should be less than ~ 1000µL

Minimum volume: 150 µL

Dimensions max.: (mm) L48 x H27 x W9,5

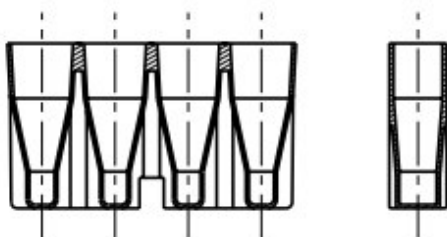
80 521 10

Example Picture of the package – 1000 Cuvettes blocks

Packaging:

1. Card Box, Dim.: (mm) 410 x 260 x 145
2. Labeling (see below)

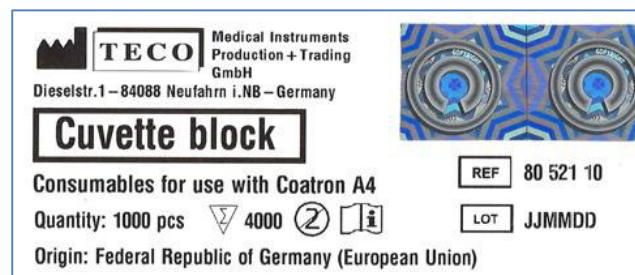Symbols key (if applicable):

Manufacturer	Lot. Number	Catalogue Number	Consult accompanying documents	Determination	Single use
--------------	-------------	------------------	--------------------------------	---------------	------------

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