

80 560 00 Cuvette block, (6pos/ea)

TECO

Information of use

The Cuvette block as general laboratory article is suitable to hold a reaction mixture for use in optical analyzers which are used in laboratories for in vitro diagnostic tests.

The cuvettes are ready for immediate use. They have unlimited shelf life if stored at 0 - 50°C.

Content

Product	Cuvette block	1000 pcs
Cat.No.	80 560 00	
Content	50 bundles (=20 blocks 6 pcs.)	6000 det.

The Cuvettes can be used with Coatron A6 (3000) Analyzer.

Depending on configuration of IVD analyser instrument a cuvette "activation"-code could be necessary. In this case the box contains an appropriate Voucher with a VIN and PIN code to generate a ticket on the web-based registration page (www.teco-reg.com).

The ticket information (VIN/PIN) must be entered for the respective device to release the number of tests for this device only. The (VIN/PIN) can only be used once per unit.

Precautions and waste information


The Cuvette should only be used once in analyzers. To prevent contamination (sample/reagent), it is advised to avoid contact with skin and eyes. Suitable protective clothing and gloves are recommended. Please also note the disposal of components in accordance with local regulations for infectious material.

Material: pure, clear Polystyrol (PS)

Maximum volume per Single use Unit should be less than ~ 1000µL

Minimum volume: 75 µL

Dimensions max.: (mm) L48 x H27 x W9,5

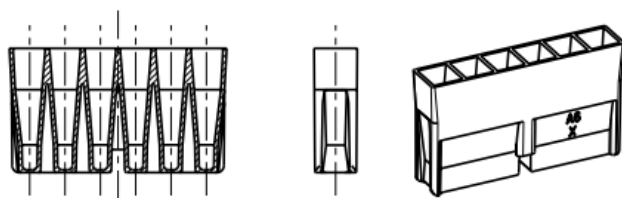

80 560 00

Example Picture of the package – 1000 Cuvettes blocks



Packaging:

1. Card Box, Dim.: (mm) 400 x 250 x 155
2. Labeling (see below)

**TECO** Medical Instruments
Production + Trading
GmbH
Dieselstr.1 – 84088 Neufahrn i.NB – Germany



Cuvette block

Consumables for use with Coatron A6

QTY: 1000 pcs  6000 

Origin: Federal Republic of Germany (European Union)

REF 80 560 00

LOT 230510

Symbols key (if applicable):

 Manufacturer	LOT Lot. Number	REF Catalogue Number	 Consult accompanying documents	 Determination	 Single use
--	------------------------	-----------------------------	--	---	--