Calcium Chloride 0,025M

TECO

Auxiliary reagent for professional use

REF

A0350-050

(EN)

Information for Use

Chemical solution intended for general laboratory use as an auxiliary reagent in various in vitro assays. Restricted to professional users.

Contents

Product	Calcium Chloride 0.025M
REF	A0350-050
CaCl ₂ 0.025M	10x5 mL

Preparation

Ready to use.

Storage and Stability

Unopened product is stable until the expiration date shown on the label stored at 2-8°C.

Opened product is stable for 30 days if stored at 2-8°C in the original vial.

Precautions

Avoid contact with skin and eyes. Wear suitable protective clothing. Dispose components in compliance with local regulations for infectious material.

Calcium Chloride Solution contains sodium azide (<0.1%) . Sodium azide under acid conditions yields Hydrazoic acid, a toxic compound. Azide compounds should be diluted with running water before being discarded. Upon disposal, azide compounds should be flushed with large volumes of water. These precautions are recommended to avoid deposits in metal pipes in which explosive conditions may develop.

Warranty

This product is warranted to perform in accordance with its labelling and literature. TECO disclaims any implied warranty of merchantability or fitness for any other purpose, and in no event will TECO be liable for any consequential damages arising out of aforesaid express warranty.

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Inhalt

Product	Calcium Chloride 0.025M
REF.	A0350-050
CaCl ₂ 0.025M	10x5 mL

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Calcium Chloride 0,025M

TECO

REF**A0350-050***Auxiliary reagent for professional use*

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