Information of Use

Chemical buffer solution intended for general laboratory use as a diluent for citrated human plasma samples in various in vitro assays. Restricted to professional users.

Content

Product	Owren's Buffer
REF	A0592-100
Content	1x100 mL

This Owren's buffer is a salt solution with 11mM tri-Sodium Citrate that is buffered to pH 7,3 with 4mM Diethyle Barbitol Buffer and with additives of 132 mM NaCl as well as 4mM Sodium Acid as a conservative.

Preparation

IBS: pH 7.2 - 7.4, liquid
Ready to use.

Storage and Stability

Unopened or opened reagents are stable until the expiration date shown on the label stored at 15-25°C. Avoid contamination and direct sun light.

Precautions

Owren's buffer contains Sodium Acid to prevent bacterial growth. Do not empty into drains Avoid contact with skin and eyes. Wear suitable protective clothing. Dispose components in compliance with local regulations for infectious material. For more information refer to the Material Safety Data Sheet (MSDS).

Warranty

This product is warranted to perform in accordance with its labelling and literature. TECO disclaims any implied warranty of merchantability or fitness for any other purpose, and in no event will TECO be liable for any consequential damages arising out of aforesaid express warranty.

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